

- Change the Intake Filters, PN 89831 once a year, or more often in dusty environments. See Figure 4.
- To comply with NFPA 99C, a 5-micron Filter is installed on all AIRSTAR models.
- Periodically inspect the Moisture Monitor. A “blue” Moisture Monitor indicates that the air in the storage tank is dry. A “pink” Moisture Monitor indicates a high level of humidity in the storage tank. To correct this situation, see TROUBLESHOOTING page 12. See Figure 5.
- The only consumable parts on the AIRSTAR are the intake filter and the 5-micron exhaust filter element. See Figure 5. When replacing either filter, dispose of the removed filter in accordance with local codes.

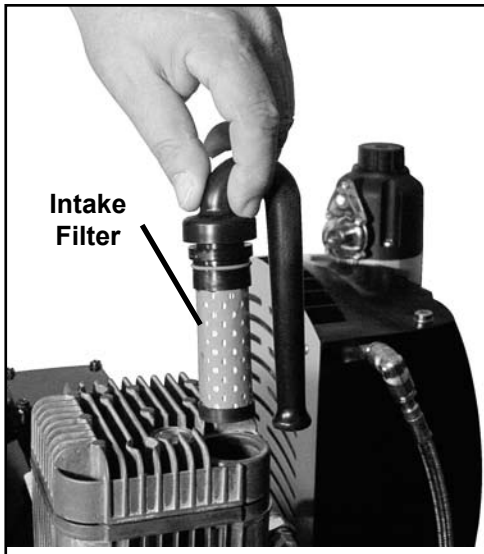


Figure 4. Intake Filter Location

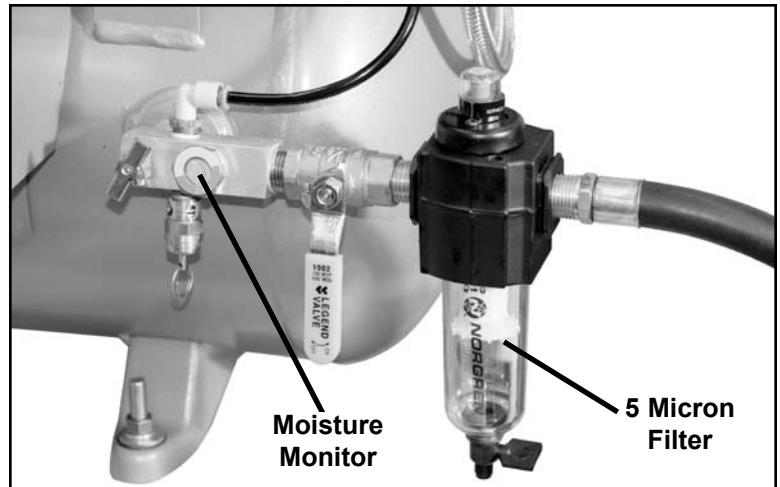


Figure 5. Moisture Monitor and 5 Micron Filter Location

OPTIONAL ACCESSORIES

Description	Model	Part Number
REMOTE AIR INTAKE KIT	AirStar 10	85491
	AirStar 21, 22, 30	85492
	AirStar 50	85493
	AirStar 70	85494
REMOTE CONTROL PANEL w/24 V switches	For all AirStars	53111
1-Switch Plate Kit		53251
2-Switch Plate Kit		53250
3-Switch Plate Kit		53133
SOUND COVER	AirStar 10	85961
	AirStar 21	85962-1M
	AirStar 22	85962-2M
	AirStar 30	85963M
	AirStar 50	89523M
AirStar 70	89574M	