



CLEANSTREAM

Dispenser System

For Use with 1 or 2 dental units.

Directions for Use:

Prepare the cleaning solution and clean the system daily by performing the following steps:

1. Fill the CleanStream Dispenser with cold tap water as applicable.
 - a. to the line marked 1L for 1 operatory
 - b. to the line marked 2L for 2 operatories
4. For first operatory cleansing, place dispenser in upright position.

2. Using the 20ml measuring line in the CleanStream cleaner bottle cap, add the CleanStream cleaner concentrate to the dispenser as applicable.
 - a. for 1 operatory, add 20ml of CleanStream concentrate to the 1 liter of water
 - b. for 2 operatories, add 40ml of CleanStream concentrate to the 2 liters of water



3. The interior of the CleanStream dispenser bottle cap is configured with three holes for the insertion of HVEs and SEs via provided adapters as follows:
 - a. attach saliva ejector to smallest atomizing adapter
 - b. attach 1 or 2 high volume ejectors to respective adapter(s)



5. With the vacuum pump on and handpiece valves open, aspirate CleanStream solution from dispenser.
6. For second operatory cleansing, place bottle in horizontal position.



7. After each cleansing procedure, disconnect the handpieces and rinse the dispenser.



CLEANSTREAM

Dispenser System

For Use with 1 or 2 dental units.

This CleanStream Starter Kit comes with the parts listed below:

Tips

PN ED-598



PN 57643
SMALL



PN ED-597



PN 57644




Cap
O-Rings
Bottle

PN 57645
PN 57647
PN 57642

* To reorder parts or entire system, refer to the Part Numbers on this Diagram.

Directions for multipurpose tip usage:

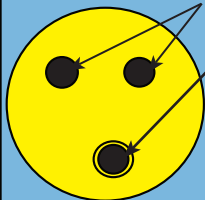

Clip tips directly into identical nests.

P/N ED-597



Use as adapter for post w/ Salvia Ejector.

*Note: Use w/ 2 HVE's

PN 57644



PN ED-598



High Volume Evacuator



Once attached these pieces stay together.

PN ED-597



Adapter shown with Cap Overview on left.

PN 57643
Small



Salvia Ejector



PN 57764 Rev. D

