

MAINTAIN YOUR VACUUM SYSTEM FOR LONGER LIFE WITH CLEANSTREAM EVACUATION SYSTEM CLEANER



CleanStream



CleanStream
by
AIR
TECHNIQUES
equipped for life

DAILY CLEANING WITH CLEANSTREAM

CleanStream cleans, deodorizes and maintains your evacuation lines optimizing system performance and keeping your vacuum system running smoothly. We recommend cleaning each operatory once a day or more often if deemed necessary due to the procedure such as extractions, periodontal therapy, or the removal of sticky resin restorations. *To ensure best cleaning results, please follow instructions below.*

1 DOSAGE 2% DILUTION - MIX & SHAKE



1 Liter



2 Liters



1 operatory: Mix 1 liter of water + 1 cap of CleanStream (20 ml) – shake lightly

2 operatories: Mix 2 liters of water + 2 caps of Cleanstream (40 ml) – shake lightly

2 CLEANING OF EVACUATION LINES – ASPIRATE



Operatory 1: Place Vortex Dispenser in upright position. Attach hoses, open handpiece valves, aspirate 1 liter per operatory (*stops automatically after 1 liter is aspirated; no monitoring needed*)


Operatory 2: Place Vortex Dispenser in horizontal position. Dispense the rest of the solution.

3 CLEANING OF CUSPIDOR (OPTIONAL)



A minimum of 250 ml of diluted CleanStream solution per cuspidor

4 DURATION TIME

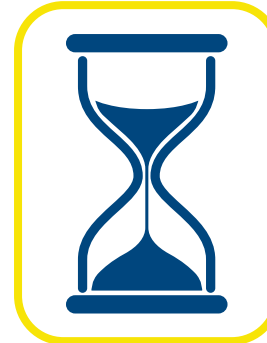

Ideally leave solution in evacuation lines for at least one hour or overnight.

5 FLUSH (OPTIONAL)



For best cleaning results, flush lines with 2 liters of water after the exposure time or the next morning prior to patient care.