
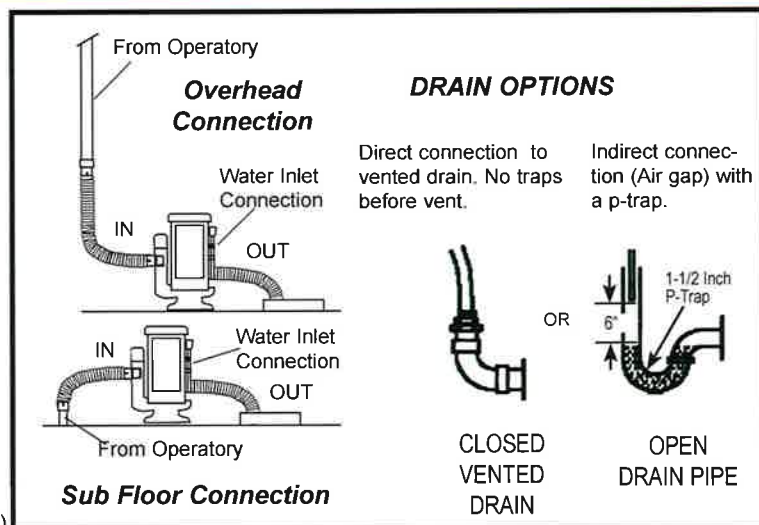


PLUMBING INSTALLATION

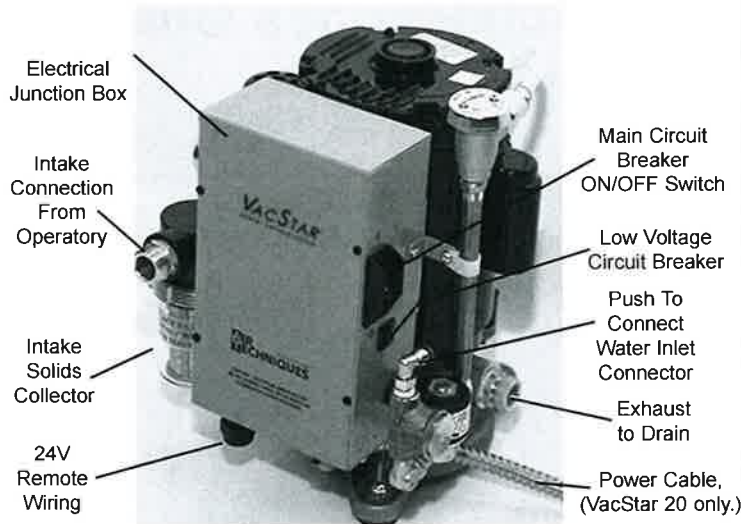


SCHEDULED MAINTENANCE

☐ Daily Maintenance - Clean vacuum lines

Flush all vacuum lines and tubing in the dental system with Monarch CleanStream Evacuation System Cleaner.


☐ Routine Inspection - Monthly

1. Check tubing for kinks or cracks.
2. Check for abnormal noises and leaks.
3. Check exterior surfaces for dirt and debris, clean if necessary.
4. Check that no flammable, corrosive, or combustible materials are stored in the equipment room (especially in the area around the equipment).
5. Refer to the right and check the vacuum relief valve filter, clean if necessary.

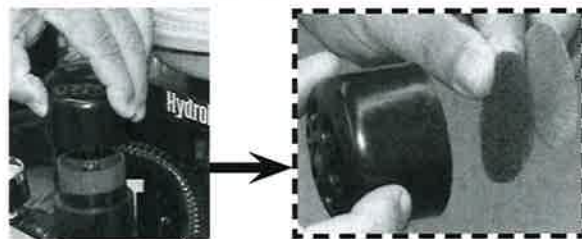

☐ Intake Solids Collector Replacement - Monthly

1. Turn off the power and water supply.
2. Unscrew the solids bowl (counter clock-wise) and remove the screen and gasket. Dispose of all three items.
3. Assemble a new bowl, screen and gasket included in the Solids Collector Replacement Kit. 55880 (55880C case of 12 kits).
4. Install the new solids collector by screwing the bowl into the solids collector body.

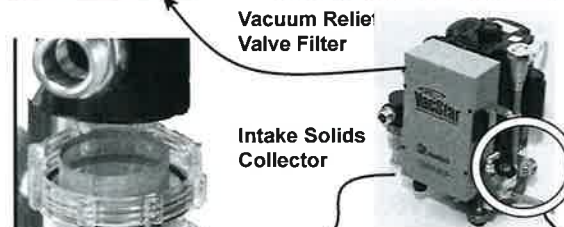

☐ Check/Clean Solenoid Water Inlet Strainer - Semi-Annually



1. Turn off the power and water supply to the equipment.
2. Use a wrench to unscrew (counter clockwise) the cover nut.
3. Remove the cover nut and strainer.
4. Inspect the strainer and clean as necessary.



Vacuum Relief Valve Filter

Intake Solids Collector

NEVER OPERATE WITHOUT THE SCREEN INSTALLED IN THE BOWL

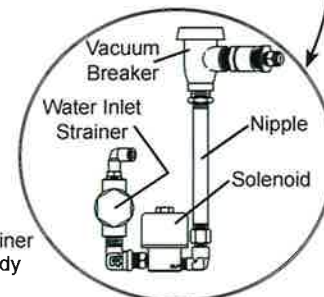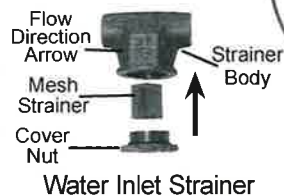

Water System Assembly

Assembling the Water Inlet Strainer

1. Orienting the assembly with the cover nut facing down as shown, seat strainer into the cover nut.
2. Insert the strainer up into the strainer body and tighten the cover nut.
3. Make sure the strainer stays perpendicular to the strainer body.
4. Push up and tighten the cover nut making sure not over tighten.

www.airtechniques.com



Corporate Headquarters
1295 Walt Whitman Road
Melville, New York 11747-3062, USA
Phone: 1-800-247-8324 Fax: 1-888-247-8481

