

Warning: Never work on equipment connected to live power. Remove power by disconnecting unit mains power prior to performing any procedures.


Multiple VacStar Pump Connections -

This section provides instructions necessary to connect up to three VacStar pumps to one remote switch.

1. Disconnect power to all pumps which are to be connected to the relay kit.
2. Connect the Remote Switch, Relay, and Pump 1 as shown in the diagram below.

Note: Pump 1 **must** remain powered (connected to mains) for all other connected pumps to turn on.

3. Connect the other pumps to the relay as shown.
4. Reconnect power to all pumps.
5. Verify that the operation of the remote switch turns **all** pumps on/off at the same time.

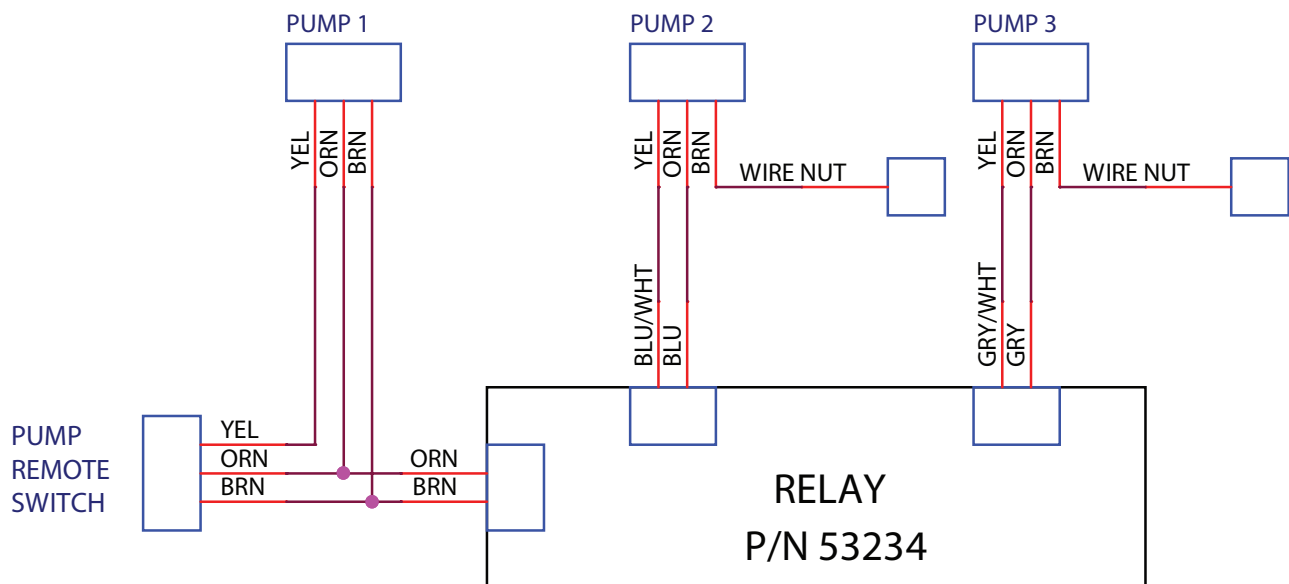

www.airtechniques.com

VACSTAR[®]

DENTAL VACUUM SYSTEM

Warning: Never work on equipment connected to live power. Remove power by disconnecting unit mains power prior to performing any procedures.

Multiple VacStar Pump Connections with Water Lockout -

This section provides instructions necessary to connect up to three VacStar pumps with the water lockout to one remote switch.


1. Disconnect power to Water System Transformer and all Pumps which are to be connected to the relay kit.
2. Connect the Water Remote Switch to the Water System Transformer as shown.

Note: The Water Remote Switch **must** be turned on before the connected pumps will turn on.


3. Connect the Pump Remote Switch, Relay and Pump 1 as shown.

Note: Pump 1 **must** remain powered (connected to mains) for all other connected pumps to turn on.

4. Connect the other pumps to the relay as shown.
5. Verify the operation of the Water Remote Switch and Pump Remote switch, and that **all** pumps turn on at the same time.

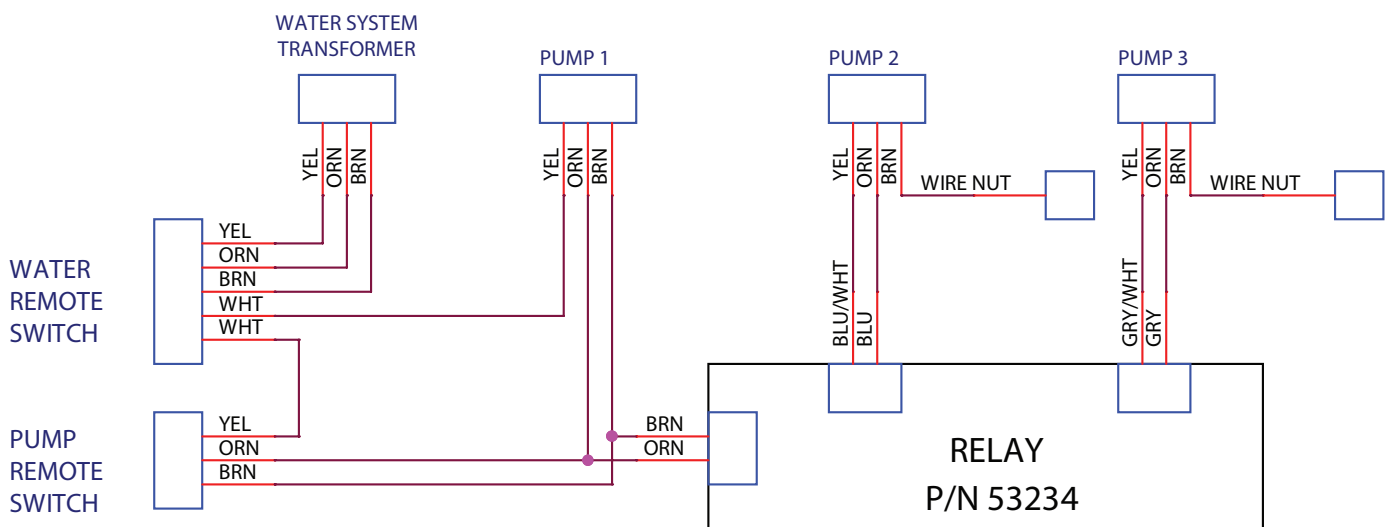


www.airtechniques.com

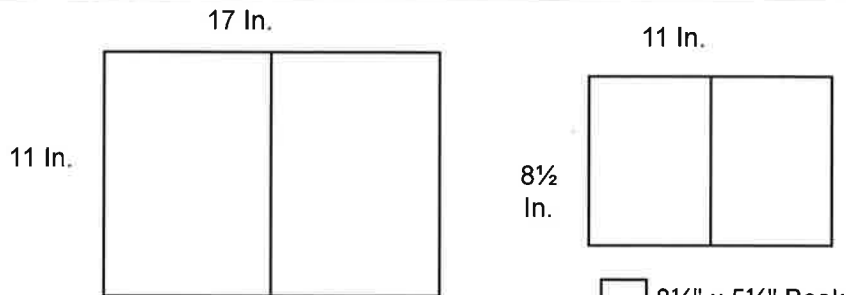


8 1/2" x 11" Book

8 1/2" x 5 1/2" Book

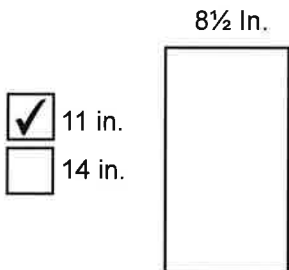

11 in.

14 in.

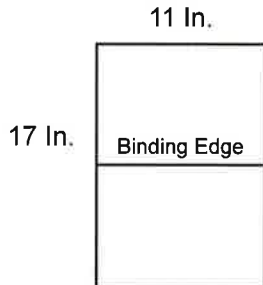

Print Single Side

Print Double Side

No Hole Punch

3-Hole @ Binding Edge

Application File Type:

- In Design
- PDF (for viewing)
- CorelDraw
- Quark XPress
- MS Word
- WordPerfect

Air Techniques Storage Location:

Stored by document part number in one of the network release drive directories checked below.

- Manuals
- Instructions

Finished Document Print Requirements

Doc size: 8.5 X11 1 page sheet
 Total sheets: 1 8.5 X11 instruction sheet

Paper requirements	Weight	Brightness	Finish
<input checked="" type="checkbox"/> Standard	24lb	98	Mat
<input type="checkbox"/> Custom			

Print Black & White

Additional Options:

- Print in Color
- Card Stock
- Saddle Stitch / Fold (Staple Binding).
- UV Coat Both Sides
- Laminate
- 3-Hole Binding
- Single hole punch left corner front side
- Spiral Bound Booklets
- Wire-O Booklets (Metal Coil Binding)

DATE	REV	REVISION RECORD	AUTH	DWN	CK
12/19	A	ECO 55-883	SFVr	RER	KML

Comments:

ORIGINAL

Artwork print file provided by Air Techniques

Important:

Make sure to print only the latest revision from the network release drive using the original document file for best printing results.
 Never print scanned copies from the (engscan network drive).

AIR TECHNIQUES equipped for life®		DRAWN	
		RER	
MATERIAL SEE RQMTS	TITLE VacStar Remote Switch Relay Kit Connection Instructions		
FINISH SEE RQMTS			
DATE 12/6/19	SIZE A	PART NO. 55889-LIT	REV A