

# CleanStream

Monarch Dispenser System



## DIRECTIONS FOR USE WITH 1 OR 2 DENTAL UNITS:

**Prepare the cleaning solution and clean the system daily by performing the following steps:**

1. Fill the Monarch CleanStream Dispenser with cold tap water as applicable.
  - a. to the line marked 1L for 1 operator
  - b. to the line marked 2L for 2 operatories
2. Using the 20ml measuring line in the CleanStream cleaner bottle cap, add the CleanStream cleaner concentrate to the dispenser as applicable.
  - a. for 1 operator, add 20ml of CleanStream concentrate to the 1 liter of water
  - b. for 2 operatories, add 40ml of CleanStream concentrate to the 2 liters of water
3. The interior of the CleanStream dispenser bottle cap is configured with 2 HVE adapters and 1 SE adapter. Attach handpiece valves as follows:
  - a. attach saliva ejector to the small adapter
  - b. attach 1 or 2 high volume ejectors to the large adapter(s)
4. For first operator cleaning, place dispenser in upright position.
5. With the vacuum pump on and handpiece valves open, aspirate Monarch CleanStream solution from dispenser.
6. For second operator cleaning, place bottle in horizontal position.
7. After each cleansing procedure, disconnect the handpieces and rinse the dispenser.

