


INSTALLATION INSTRUCTIONS

AIRSTAR REMOTE AIR INTAKE KITS: 85491, 85492, 85493, 85494, 87361, 85511

This illustration is for Airstar 50. Other models install in similar manner.

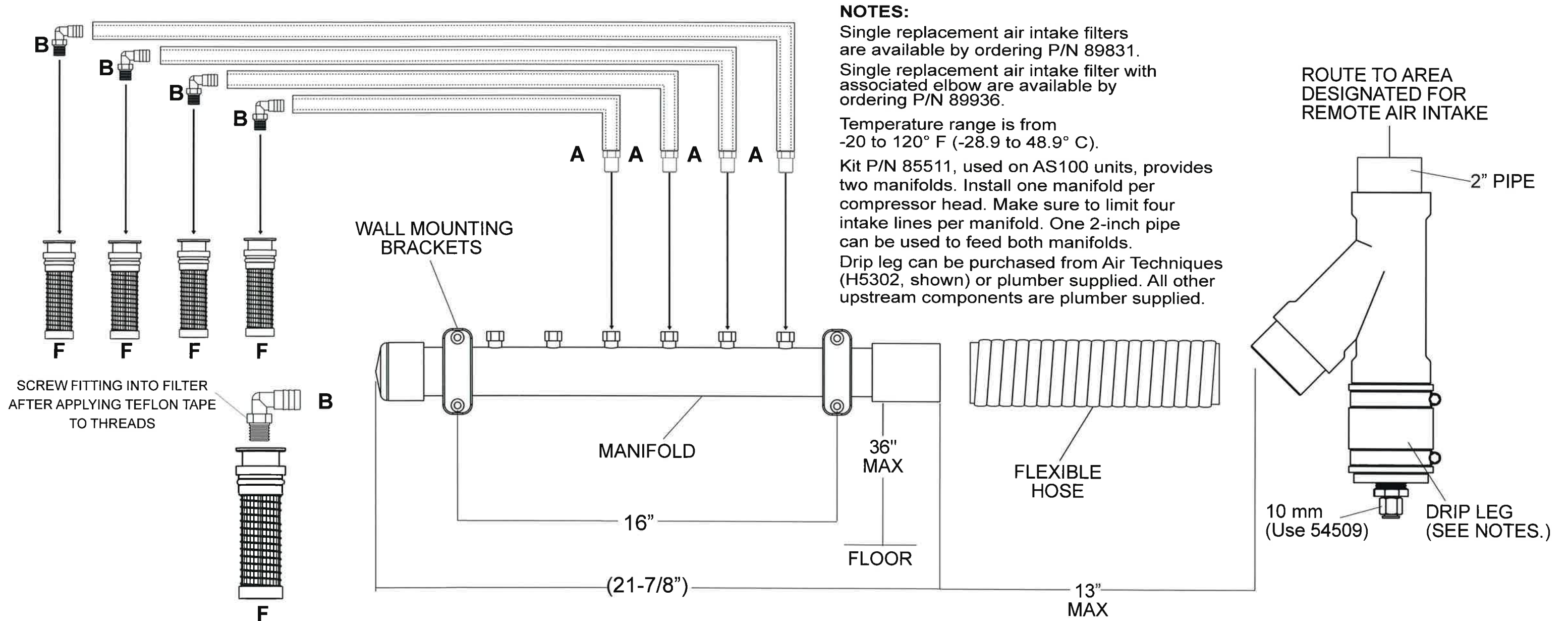

Installation Instructions:

1. Mount manifold on wall using brackets and hardware supplied.
2. Connect manifold to 2-inch plumber supplied pipe using flexible hose and clamps supplied.
3. Remove plug(s) from manifold and install hoses "A". Need one hose "A" per each AirStar intake filter.
4. Connect each hose "A" to each fitting "B".
5. Insert each fitting "B" into each intake filter "F" on the AirStar.

ORIGINAL

Air Techniques Inc.
1295 Walt Whitman Road
Melville, NY 11747

www.airtechniques.com

P/N 85504 Rev. M, September 2019