

Contents

Part No.	Description	Quantity
87169	Hose; 3/8" I.D., 3/8" MNPT X 3/8" MNPT Swivel	1
87186	PCONN Nipple Close, 3/8" MNPT	1
87168	Filter Assembly, 3/8"	1
58017R-1	PCONN Reducing Bushing 1/2" MNPT X 3/8" FNPT	1
31929	PCONN Reducing Coupling 1/2" FNPT X 3/8" FNPT	1

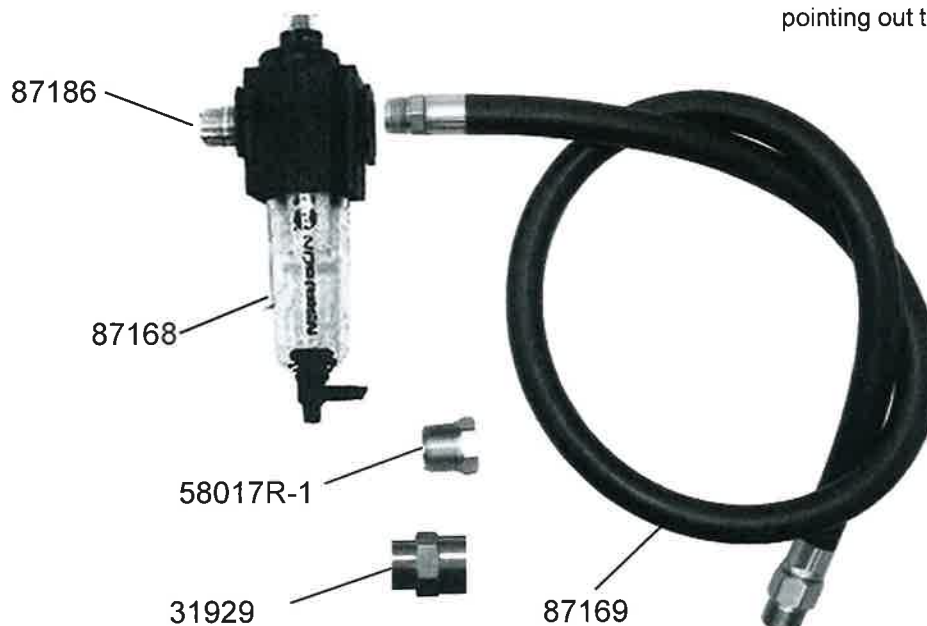
Note: Use pipe sealant or Teflon tape on all NPT connections.

1. Screw Pconn Nipple (87186) into inlet port of filter.

Note: Make sure to remove the filter bowl before installing the filter body.

2. Remove the bowl of the filter assembly (87168) and place to the side for later installation.
3. Screw filter assembly (87168) into outlet of 1/4 turn valve in tank outlet.
4. Install the bowl to the filter assembly (87168).
5. Screw Hose (87169) into outlet port of filter.
6. Depending on the operatory pipe connection type (FNPT or MNPT), perform either step a. or step b. to connect the corresponding reducing bushing.
 - a. If operatory connection is 1/2" FNPT, connect the 1/2" reducing bushing (58017R-1).
 - b. If operatory connection is 1/2" MNPT, connect the 1/2" reducing bushing (31929).
7. Connect the swivel end of hose (87169) to the associated PCONN (58017R-1 or 31929) installed by step 6 above.

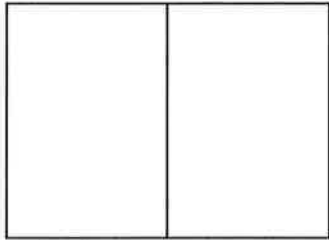
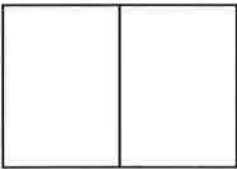
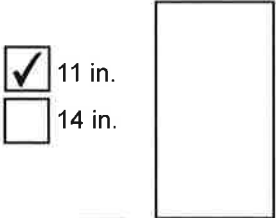
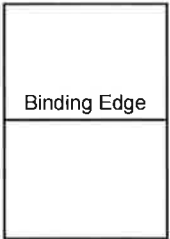
Note: Make sure to install the filter with arrow pointing out towards the hose assembly.



ORIGINAL

B

B

17 In.	11 In.
	
☐ 8 1/2" x 11" Book	☐ 8 1/2" x 5 1/2" Book
11 In.	11 In.
	
☒ 11 in.	
☐ 14 in.	
☒ Print Single Side	☐ No Hole Punch
☐ Print Double Side	☐ 3-Hole @ Binding Edge

DATE	REV	REVISION RECORD	AUTH	DWN	CK
10/18	H	ECO 85-1373	SFJr	RER	

Comments:

ORIGINAL

Application File Type:

- | | |
|---|---------------------------------------|
| ☒ In Design | ☐ Quark XPress |
| ☒ PDF (for viewing) | ☐ MS Word |
| ☐ CorelDraw | ☐ WordPerfect |

Air Techniques Storage Location:

Stored by document part number in one of the network release drive directories checked below.

- ☐
- Manuals
- ☒
- Instructions

Finished Document Print Requirements

Doc size: 8.5 X 11 1 page Sheet

Total sheets: 1 11 x 8.5 sheet

Paper requirements	Weight	Brightness	Finish
☒ Standard	24lb	98	Mat
☐ Custom			

Print Black & White

Additional Options:

- ☐
- Print in Color
-
- ☐
- Card Stock
-
- ☐
- Saddle Stitch / Fold (Staple Binding).
-
- ☐
- UV Coat Both Sides
-
- ☐
- Laminate
-
- ☐
- 3-Hole Binding
-
- ☐
- Single hole punch left corner front side
-
- ☐
- Spiral Bound Booklets
-
- ☐
- Wire-O Booklets (Metal Coil Binding)

Artwork print file provided by Air Techniques

Important:

Make sure to print only the latest revision from the network release drive using the original document file for best printing results.
 Never print scanned copies from the (engscan network drive).

AIR TECHNIQUES equipped for life®		DRAWN RER	
MATERIAL SEE RQMTS	TITLE AirStar Filter Kit Installation Instructions		
FINISH SEE RQMTS			
DATE 10/17/18	SIZE A	PART NO. 86394	REV H

A

A