


# Touch Screen Replacement Kit

Part No: 87284

## Replacement and Calibration Instructions


### Touch Screen Replacement.

1. Set the power switch to OFF and disconnect the unit from mains power.
2. Open the front panel of the device.
3. Disconnect the ribbon cable from the back of the screen. Take care to avoid breaking the alignment tab.
4. Remove the nut indicated by the blue arrow. If applicable, remove the existing touch screen ground cable from the stud.
5. Remove the four screws and washers indicated by the red arrows, setting them aside for use on the new touch screen.
6. Remove the touch screen (86438) to be replaced.
7. Install the replacement touch screen and secure with the four screws and washers removed previously.
8. Attach the touch screen ground wire to the stud and secure with nut torqued to 35 in-lb.
9. Connect the ribbon cable to the screen, aligning the tab on the cable with the notch on the connector.
10. Connect the unit to mains power and set the power switch to ON. Verify the screen properly displays images.
11. Perform the calibration process listed below.

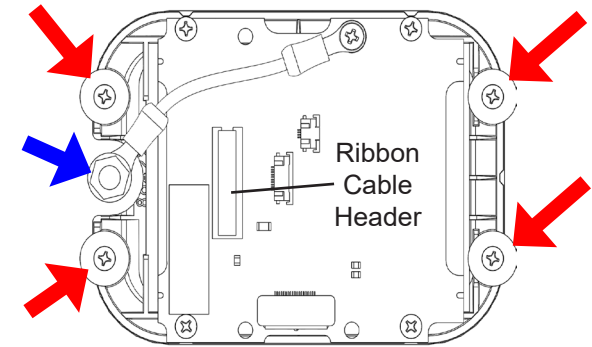

### Calibration Process:

1. Start by touching and holding a finger on the screen shown below for about ten seconds or until the first image appears on the screen.

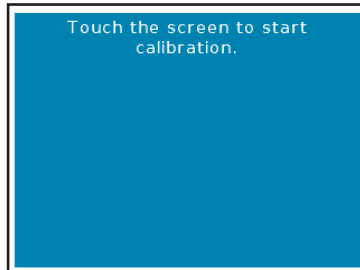

2. Follow the on screen prompts shown to the right. Use a precise but **dull** item such as a plastic stylus or no-point Phillips head screw driver. Do not use a utility knife, tack, paperclip, sharp tipped screw driver or other sharp tip item. Be sure to touch as close to the center of the red dots as possible.
3. Confirm each touchscreen response.

