

For Use with 1 or 2 dental units.

Directions for Use:

Prepare the cleaning solution and clean the system daily by performing the following steps:

- **1.** Fill the Monarch CleanStream Dispenser with cold tap water as applicable.
 - a. to the line marked 1L for 1 operatory
 - b. to the line marked 2L for 2 operatories
- 2. Using the 20ml measuring line in the CleanStream cleaner bottle cap, add the CleanStream cleaner concentrate to the dispenser as applicable.
 - a. for 1 operatory, add 20ml of CleanStream concentrate to the 1 liter of water
 - b. for 2 operatories, add 40ml of CleanStream concentrate to the 2 liters of water





- The interior of the CleanStream dispenser bottle cap is configured with three holes for the insertion of HVEs and SEs via provided adapters as follows:
 - a. attach saliva ejector to smallest atomizing adapter
 - b. attach 1 or 2 high volume ejectors to respective adapter(s)



4. For first operatory cleaning, place dispenser in upright position.



- With the vacuum pump on and handpiece valves open, aspirate Monarch CleanStream solution from dispenser.
- **6.** For second operatory cleaning, place bottle in horizontal position.



 After each cleansing procedure, disconnect the handpieces and rinse the dispenser.





CleanStream Dispenser System

For Use with 1 or 2 dental units.



