

# MAINTAIN YOUR VACUUM SYSTEM FOR LONGER LIFE WITH MONARCH CLEANSTREAM EVACUATION SYSTEM CLEANER



# CleanStream



 **Monarch.**  
by  
**AIR** **TECHNIQUES**  
equipped for life®

# DAILY CLEANING WITH MONARCH CLEANSTREAM

Monarch CleanStream cleans, deodorizes and maintains your evacuation lines optimizing system performance and keeping your vacuum system running smoothly. We recommend cleaning each operatory once a day.

To ensure best cleaning results, please follow instructions below.

## 1 PRE-CLEANING



Run 2 liters of water through the lines

## 2 DOSAGE 2%



**1 operatory:** 1 liter of water + 1 cap of Monarch CleanStream (20 ml)

**2 operatories:** 2 liters of water + 2 caps of Monarch Cleanstream (40 ml)

## 3 CLEANING OF EVACUATION LINES



OPERATORY 1

OPERATORY 2

**Operatory 1:** Place Dispenser in upright position. Attach hoses, open handpiece valves, aspirate 1 liter per operatory (stops automatically after 1 liter is aspirated; no monitoring needed)


**Operatory 2:** Place Dispenser in horizontal position. Dispense the rest of the solution

## 4 CLEANING OF CUSPIDOR (OPTIONAL)



A minimum of 250 ml of diluted CleanStream solution per cuspidor

## 5 DURATION TIME

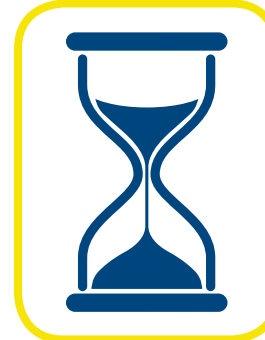

For optimal cleaning results, leave solution in lines for at least an hour or overnight

## 6 FLUSH



Flush lines with 2 liters of water after the exposure time or the next morning

 **Monarch**  
by  
**AIR TECHNIQUES** equipped for life®