

CleanStream Waterlines Cleaner

Important: Since thick slime is discharged during first time cleaning sessions, remove handpieces etc. to avoid plugging and clogging.

Disconnect all water heaters. Identify and disconnect all dead waterline legs including slow speed Waterlines.

Note: For optimal cleaning, air purge and clean all instruments and tubing that contain water using step 1, below, including but not limited to: handpiece, air/water syringe, scaler, quick disconnect line.

Directions for Use:

Step 1: Air Purge Waterlines

1. Turn unit OFF. Empty water bottle and place back onto the unit.
2. Turn unit ON.
3. Air purge Waterlines by activating handpiece hoses and air/water syringes until water is eliminated.

Step 2: Add CleanStream Waterlines Cleaner using the included measuring cup.

1. Turn unit OFF. Remove water bottle, measure 2 ounce of CleanStream Waterlines Cleaner using the measuring cup and pour into the unit's empty water bottle. Swish CleanStream Waterlines Cleaner in bottle and then place water bottle onto the unit.
2. Turn unit ON. Activate handpiece hoses and air/water syringes until CleanStream Waterlines Cleaner exits.
3. Turn unit OFF. Allow CleanStream Waterlines Cleaner to treat waterlines and bottle for at least 6 hours before proceeding to Step 3.

Step 3: Flush Waterlines

1. Remove water bottle, dispose of any excess CleanStream Waterlines Cleaner and fill bottle with water source.
2. Turn unit ON. Flush waterlines with water for 30 seconds to remove any excess Monarch Waterline Cleaner.
3. Record task completion on System Cleaning Task Log provided. Continue with procedures as necessary.

Storage and Disposal:

Refer to bottle label for storage and disposal information.

Corporate Headquarters
1295 Walt Whitman Road | Melville, New York 11747- 3062
Phone: 800-247-8324 | Fax: 888-247-8481

www.airtechniques.com

