



10 PREV EVALS 24 YEARS EXP

I loved the ease of using the hands-free dispenser.

Stacey Abbatessa, RDH Danvers, MA



5 PREV EVALS 20 YEARS EXP

It was easy with the hookups and saved us time mixing and flushing the lines.

Steven Berk, DDS Annandale, VA



7 PREV EVALS 28 YEARS EXP

The hygiene operator traps were much cleaner after 3 weeks of use.

Cathy Hendrickson, RDH Greeley, CO

AIR TECHNIQUES

Monarch CleanStream and Dispenser System

4.4

Dental PRODUCT SHOPPER BEST PRODUCT

Keep evacuation lines clean and debris-free with a proprietary formula that's easy to use with a new hands-free, easy-to-use dispenser

The work day is coming to a close and Stacey Abbatessa is checking off items on a long list of end-of-day duties she needs to complete before she can leave the office and get home to her kids. Roughly 113 miles to the west, Kristen Khoury embarks on a similar mission to "tidy up a million things" before she can walk out the door and call it a day.

Like most hygienists, Abbatessa and Khoury are always looking for systems to streamline their end-of-day duties and help them use their time more efficiently. Ironically, it was by adding one more duty that these 2 hygienists can now end their work days more quickly and efficiently. The task? Evaluating the Monarch CleanStream and Dispenser System for *Dental Product Shopper*.

The state-of-the-art Monarch CleanStream system from Air Techniques cleans and protects evacuation lines with a proprietary formula, and a new hands-free dispenser improves efficiency. "CleanStream exceeded my expectations with its ease of use, and I loved the scent," shared Abbatessa, who especially "loved the ease of using the hands-free dispenser, which let me continue disinfecting and processing instruments without worrying about being tied up flushing my lines."

Using CleanStream "saves me a few minutes at the end of the day," shared Khoury. "Hands-free dispensing rocks, and I don't have to monitor it while doing other things." Abbatessa is thrilled that she "can fill the dispenser and walk away to continue other end-of-day tasks, knowing that the job is being done effectively."



Ease of Use

Compatible with both wet and dry vacuum systems, CleanStream yields 125 cleanings per bottle and mixes easily with water. Users simply fill the dispenser with water to the line, measure the solution in the bottle cap, add the concentrate to the water, and attach the saliva ejector and HVE hoses. For Dr. Steven Berk, CleanStream made chairside evacuation maintenance "easy with the hookups and saved us time mixing and flushing the lines." With 6 operatories, Dr. Carrie Niemann found it "simple to premix for 2 rooms at a time," and said, "It works well to attach and clean multiple lines at a time."

"I like the way the cap is designed, and the straws reach all the way to the container's bottom, sucking up every last bit of solution," said Khoury. Because the dispenser has a sealed cover, she can make enough solution for 2 days at once, which she called "a little timesaver." Khoury suggested making the "full" marking visible all the way around the container to make it easier to see when enough water has been added.

takeaways

- Optimizes suction and prevents debris buildup for uninterrupted performance and extended equipment longevity
- Vortex Technology ensures optimal cleaning throughout the inner tubing walls
- Compatible with any dental vacuum system
- Hands-free dispenser maximizes ease of use and efficiency
- Non-corrosive and non-foaming formula has a neutral pH and pleasant scent



Sarah Hester, RDH Tucson, AZ

"I noticed an increase in suction power after the first use."

Suction & Cleaning

Used only once a day, the non-corrosive, non-foaming, neutral pH CleanStream formula keeps evacuation lines clean and debris-free, optimizing suction performance, preventing shutdowns, and extending equipment longevity. The proprietary blend of surfactants and chelating agents removes debris and prevents buildup in both wet and dry vacuums, while Vortex Technology creates an air-liquid mixture that atomizes the solution through the lines, fully cleaning the inner tubing walls.

CleanStream "restored slow lines to normal suction power" for Sarah Hester, who "noticed an increase in suction power after the first use." Noting that her "hygiene operator traps were much cleaner after 3 weeks of use," Cathy Hendrickson, RDH, appreciated "not having to stop during a surgery to unclog a line," and said the dental assistants in her practice no longer report clogged suction. Dr. Berk said his evacuation system "smells and works better," and Dr. Sylvia Irwin reported, "The suction worked really well. The product was great!"

Pleasant Scent

Comments about CleanStream's pleasant scent were overwhelmingly positive, with Hendrickson calling it "very simple to use with its container

with a nice scent." Dr. Irwin said, "The evacuation system was clean, and the smell was great." Having used a previous product with a "horrible" smell, dental assistant Nicole Burke Hodges called CleanStream's scent "very pleasant." Khoury praised its "pleasant lemony scent," while Dr. Niemann said, "The smell was pleasant."

Overall Satisfaction

Exceptional ease of use and time savings, combined with CleanStream's efficacy and pleasant scent, were the


source of rave reviews from the *DPS* evaluators. "I would recommend this product due to the time it saved and the ease of using it," noted Dr. Berk. "I wouldn't change a thing" about the Monarch CleanStream and Dispenser System, concluded Abbatessa. "The ease of use, fresh scent, and delivery system make the product sell itself."

FOR MORE INFORMATION:

800.247.8324
www.airtechniques.com

PRODUCT EVALUATION SNAPSHOT

CRITERIA BASED ON AVERAGE SCORE (OUT OF 5)

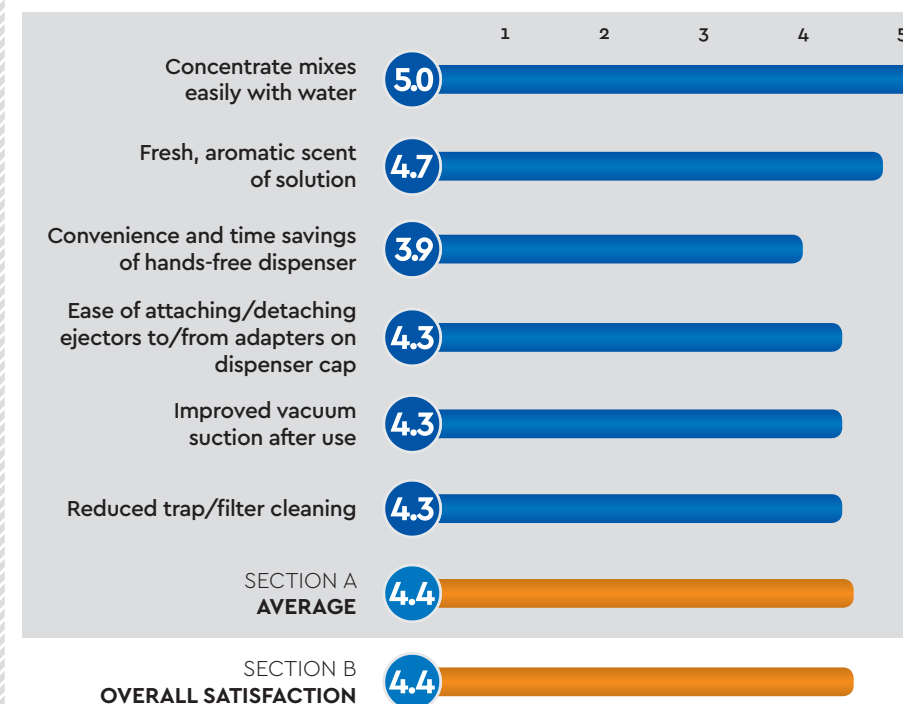

AIR TECHNIQUES
MONARCH CLEANSTREAM AND DISPENSER SYSTEM

FINAL SCORE
AVERAGE OF SECTIONS A AND B
4.4

