


# ProVecta PROTECTIVE BITE BLOCK COVER INSTALLATION

Intended Use: Provecta Protective Bite Block Covers provide a barrier to prevent cross contamination between patients when using the Bite Block for patient head alignment.

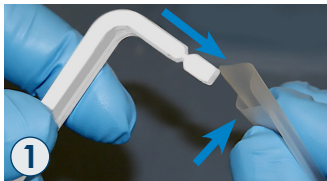

Wearing gloves, open cover by sliding between the thumb and forefinger, Insert Bite Block



Pull the cover completely onto the Bite Block



Install the Bite Block with Cover into Chin Rest



One Time Use



Suzhou Pac-Dent Technology Co., Ltd.  
18, Yuanqi Road, Suzhou City  
215133 Jiangsu, China

PN A7377 Rev. C [www.airtechniques.com](http://www.airtechniques.com)

Manufactured for/Distributed by:

**AIR**  
**TECHNIQUES** equipped for life®