



WARNING: Replace the main bolts in #1 one at a time. Do not remove both bolts at the same time.

- 1 Locate bolts under the seat upholstery. Replace the bolts with the longer bolts in the kit.
- 2 Install the GCS Mounting Bracket to the chair.
- 3 Remove the two bottom screws holding the Mixed Bag Tee to the GCS.
- 4 Install the GCS conversion bracket to the front plate, overlapping the empty holes exposed by removal of the 2 screws.
- 5 Assemble GCS with mixed Bag Tee and Scavenger to GCS bracket as shown.
- 6 Attach mixed gas and scavenger hoses to back of the GCS









