

SCAN▶X®

Barrier Envelopes

AIR
TECHNIQUES

Barrier Envelopes - Ordering Information

Plate Size	Qty.	Air Techniques P/N
#0	100	73248-0
#1	100	73248-1
#2	300	73248-2
#2	1000	73248-2K
#3	100	73248-3
#4	50	73248-4


Intended Use: Disposable Barrier Envelopes are intended to be used as a disposable barrier for Air Techniques Phosphor Storage Plates. This device is non-sterile and intended for single patient use only.

Description: ScanX Barrier Envelopes are designed for use with Air Techniques' intraoral Phosphor Storage Plates. The following pictures can be used as a guide, however, it is important to refer to the ScanX Digital Imaging System Operator's Manual for complete instructions on use of the ScanX and all accessories.

- 1 Insert the Phosphor Storage Plate.



- 2 Peel off the white adhesive strip.

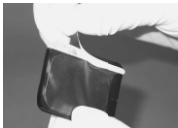


3 Seal the Barrier Envelope.



Clean the Barrier Envelope after being removed from the patient's mouth with a disinfectant wipe, such as Air Techniques Monarch Surface Wipes. Allow the Barrier Envelope to completely dry prior to ejecting the PSP.

4 Tear open the Barrier Envelope to eject the Plate after use.

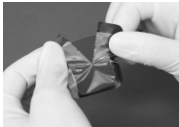

Dispose of Barrier Envelopes in accordance with all local regulations.

Do not use a Barrier Envelope that has been punctured or torn.

The user/patient is required to report to the manufacturer and the competent authority of the Member State in which the user and/or patient is established any serious incident that has occurred in relation to the device.



Only for use with ScanX



TEMP + HUMIDITY



Medical Device Safety Service
Schiffgraben 41
30175 Hannover, Germany

Australian Sponsor: Emergo Australia
Level 20 Tower II, Darling Park
201 Sussex Street, Sydney, NSW 2000, Australia



Air Techniques, Inc.
1295 Walt Whitman Road
Melville NY 11747, USA
www.airtechniques.com