
SCANX® Barrier Envelopes

Ordering Information

| | | |
|-------|--------|-----------|
| 100x | Size 0 | G8511-0 |
| 100x | Size 1 | G8511-1 |
| 300x | Size 2 | G8511-2 |
| 1000x | Size 2 | G8511-2-k |
| 100x | Size 3 | G8511-3 |
| 50x | Size 4 | G8511-4 |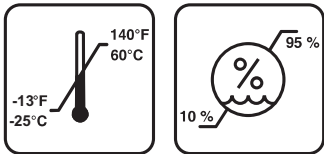

Australian Sponsor: Emergo Australia
Level 20 Tower II, Darling Park
201 Sussex Street, Sydney, NSW 2000
Australia

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EN Barrier envelope

Intended use

Disposable Barrier Envelopes are intended to be used as a disposable barrier for Air Techniques Phosphor Storage Plates. This device is non-sterile and intended for single patient use only.

Description: ScanX Barrier Envelopes are designed for use with Air Techniques' intraoral Phosphor Storage Plates. It is important to refer to the ScanX Digital Imaging System Operator's Manual for complete instructions on use of the ScanX and all accessories.

- Always choose the appropriate size.
- Completely slide the PSP into the barrier envelope. The black (inactive) side of the PSP must be visible.
- Pull off the adhesive strip and close the barrier envelope tightly by pressing together firmly.

Clean the Barrier Envelope after being removed from the patient's mouth with a disinfectant wipe, such as Air Techniques Monarch Surface Wipes. Allow the Barrier Envelope to completely dry prior to ejecting the PSP.

Dispose of Barrier Envelopes in accordance with all local regulations.

Do not use a Barrier Envelopes that has been punctured or torn.

The user / patient is required to report to the manufacturer and the competent authority of the Member State in which the user and/or patient is established any serious incident that has occurred in relation to the device.

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