

Dental Unit Waterlines Maintenance Protocol Utilizing Monarch Lines

After the start up shock protocol utilizing Monarch Lines has been completed, _____ (Practice Name) will utilize Monarch Lines weekly as the waterline maintenance product.

The day of the week that each operatory bottle will have Monarch Lines dispensed through the waterlines is _____.

Maintenance Protocol:

Step 1: Air Purge Waterlines

- Turn unit off
- Empty water bottle and place back onto the unit
- Turn unit on
- Air purge waterlines by activating handpiece hoses and air/water syringes until water is eliminated.

Step 2: Add Monarch Lines Cleaner

- Turn unit OFF. Remove water bottle, pour 2 ounces of Monarch Lines Cleaner into the unit's empty water bottle and reattach the bottle to the unit with undiluted product in the bottle.
- Turn unit ON.
- Activate Air Water Syringe/ Ultra Sonic / High Speed Handpiece/ and any line attached to the dental unit water bottle supply.
- Turn unit OFF.
- Allow Monarch Lines Cleaner to remain in the lines undisturbed for at least 6 hours, overnight or over the weekend before proceeding to Step 3. Monarch Lines can safely be left in the dental unit and have full effectiveness for up to 1 week.
- Record task completion in the log.

Step 3: Flush Waterlines

- Remove water bottle, dispose of any excess Monarch Lines Cleaner, rinse the water bottle and fill bottle with clean potable water (tap water, distilled water, or filtered bottled water) that meets the EPA Safe Drinking Water Act Standard.
- Turn unit ON. Flush each treated waterline with water for a minimum of 30 seconds until the water is clear to remove excess Monarch Lines Cleaner.



