

CamX Spectra



J2001

Current version of the manual is available at <https://bit.ly/m/air-techniques-imaging-manuals>



Caries view:
 > Carry out teeth cleaning.



Prophylaxis view:
 > **Don't** carry out teeth cleaning.



DÜRR DENTAL SE

Höpfungheimer Str. 17
 74321 Bietigheim-Bissingen, Germany

1

2


3

4

5

6

Analysis

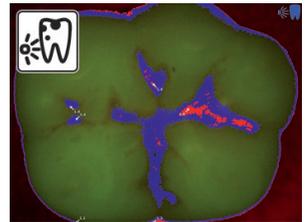

	Healthy tooth enamel
	Early-stage caries, incipient enamel caries
	Enamel caries up to enamel/dentin junction
	Dentin junction already exceeded
	Dentin caries

Use gold standard techniques to examine for potential caries.



Air Techniques, Inc.
 1295 Walt Whitman Road
 Melville, New York 11747-3062, USA



CamX Spectra



J2001

La versión actual del manual está disponible en <https://bit.ly/m/air-techniques-imaging-manuals>



Para la vista caries:
 > Realizar una limpieza.







Para la vista profiláctica:
 > **No** realizar una limpieza.




DÜRR DENTAL SE

Höpfheimer Str. 17
 74321 Bietigheim-Bissingen, Germany

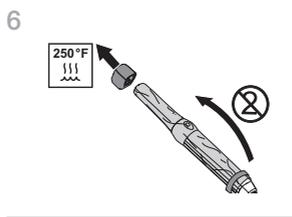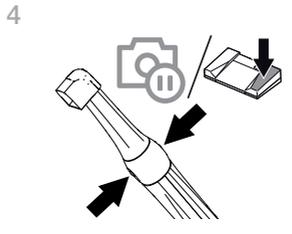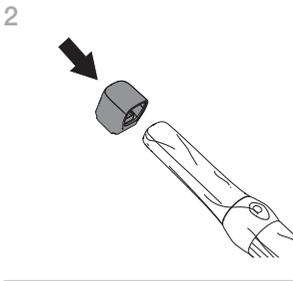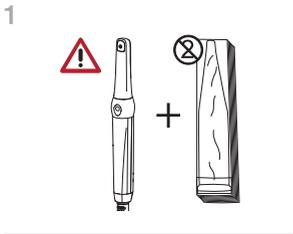

Evaluación

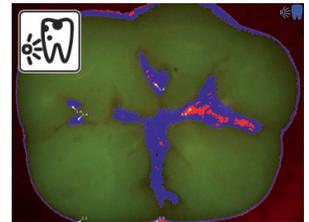

	Esmalte dental sano
	Caries inicial, caries inicial del esmalte
	Caries de esmalte hasta la frontera dentina esmalte
	Límite de dentina ya superado
	Caries en la dentina

Uso de pruebas estándar para el estudio de potenciales lesiones cariosas.



Air Techniques, Inc.
 1295 Walt Whitman Road
 Melville, New York 11747-3062, USA

