

AIR TECHNIQUES

# CleanStream: Peace of Mind for Your Operatory Lines

CleanStream's ready-to-use, hassle-free waterline cleaner and rapid in-office test kit give clinicians assurance that their dental unit waterlines are free of microbial contamination

**E**ver wonder what's lurking in your dental unit waterlines? Perhaps it's time to gain peace of mind with the CleanStream Waterlines Cleaner and Waterline Test Kit from Air Techniques. The noncorrosive, premixed cleaning formula and rapid in-office waterline testing kit remove the difficulty and uncertainty from waterline maintenance and replace it with confidence that waterlines are free of microbial contamination.

The clinical team at the practice of Jeffrey Briney, DDS, has trusted the CleanStream Waterlines Cleaner and CleanStream Waterline Test Kit to clean and maintain their waterlines for more than 5 years.

DPS spoke with hygienists Karrie Thomson and Angie Faletta, who manage the waterlines in the hygiene operatories at the Dana Point, CA, practice, along with dental assistant Angie Burkett, who oversees the lines in the doctor's operatories. Together, they shared why they believe every dental practice should adopt the CleanStream Waterlines Cleaner and Waterline Test Kit.



## IMMEDIATE ASSURANCE

When Dr. Amir Noori first began using the CleanStream Waterlines Cleaner over 2 years ago in his Huntington Beach, CA, practice, he had the water's biofilm levels tested at a lab before and after using the cleaner. "The results showed a drastic drop in biofilm after using, bringing [bacteria levels] to a minimal and safe level," he reported.

Now, he trusts the "simple, quick, and efficient" in-office CleanStream Waterline Test Kit to warn him of any microbial contamination.

"The kit provides a streamlined and convenient testing method that is both time- and cost-efficient," he shared. "It won't take too much time away from patient care and eliminates the need to send samples to a lab for testing."




1 Collect and filter sample



2 Wash sample with saline



3 Add sample to test vial

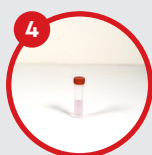

4 Wait 15 minutes for results

## A Simple Routine

The CleanStream Waterlines Cleaner was extremely easy to implement in our office, especially since our operatories are self-contained with incorporated bottle systems. It's so easy to use that a new assistant who joined our team—someone who had zero knowledge of the system—was able to acclimate on the first day.

After removing the water from the operatory's bottle system, we just pour in the premixed CleanStream Waterlines Cleaner solution, run all the lines for a couple of seconds, and then let it sit for the weekend.

On Monday, we flush out the lines with water and immediately sit the patient. Each operatory never takes more than 3 to 4 minutes to clean weekly, and we also maintain the lines on a daily/nightly schedule.

## Clean Lines, Clear Minds

In our practice, patient safety is top priority. Prior to using CleanStream Waterlines Cleaner, we were shocked when water tests revealed high bacteria counts in our waterlines, despite having new bottle-contained delivery units and consistently using sterile and distilled water in our operatories. Those unexpected results motivated us to explore solutions to get our waterlines back to levels that exceeded standard of care. CleanStream made that process simple and straightforward.

With CleanStream Waterlines Cleaner, it is very comforting to know that our lines are free of any residual bacteria and that we're maintaining a safe environment for our patients. The in-office, 15-minute CleanStream Waterline Test Kit

provides immediate reassurance that our lines are safe and compliant. We also perform regular water tests that are sent to a testing facility, and we pass with flying colors every single time.

## Proven Results

We believe every dental practice that cares about the safety of their patients should be using the CleanStream Waterlines Cleaner and CleanStream Waterline Test Kit. It has been proven over and over again in our office. Having the ease of mind when we leave the office every day that we provided a safe environment for our patients is indescribable.

## SAY 'GOODBYE' TO CLOGGED LINES & DOWNTIME

CleanStream Evacuation System Cleaner is carefully formulated to help optimize evacuation system performance and keep vacuum systems running smoothly. Sarah Hester, RDH, said she "noticed an increase in suction power after the first use," and the cleaner "restored slow lines to normal suction power."

Here are 5 reasons to give the system a try:

- 1 Penetrates and removes line buildup for improved suction performance
- 2 Cleans and deodorizes evacuation lines for all wet and dry vacuum systems
- 3 Nonfoaming, nonenzymatic concentrated formula lasts up to 5 days premixed
- 4 Noncorrosive, biodegradable, and nonoxidizing solution with a pH of 7-8
- 5 Innovative CleanStream dispenser atomizes cleaning solution to clean inner walls of tubing for mess-free and spill-free mixing and cleaning

