

Acceptance Test Phantom 3D Set acc DIN 6868-161



EN-US Quick Reference Guide

The latest version of the instructions is available in this page:



<https://bit.ly/m/air-techniques-imaging-manuals>

Contents



Important information

| | | |
|----------|--------------------------------------|---|
| 1 | About this document | 2 |
| 1.1 | Warnings and symbols | 2 |
| 1.2 | Copyright information | 2 |
| 2 | Safety | 3 |
| 2.1 | Transportation and storage | 3 |



Product description

| | | |
|----------|-----------------------------|---|
| 3 | Overview | 4 |
| 3.1 | Scope of delivery | 4 |
| 3.2 | Accessories | 4 |




Usage

! Important information

1 About this document

These installation and operating instructions are an integral part of the unit.

 The manufacturer and the distributor accept no responsibility or liability for safe operation and reliable functioning of the device if the information and instructions contained in these installation and operating instructions are not observed.

The German version of the installation and operating instructions is the original manual. All other languages are translations of the original manual.

1.1 Warnings and symbols

Warnings

The warning notes in this document highlight possible injury to persons or damage to machinery. They are marked with the following warning symbols:



General warning symbol

The warnings are structured as follows:



SIGNAL WORD

Description of type and source of danger

Here you will find the possible consequences of ignoring the warning

- › Follow these measures to avoid the danger.

The signal word differentiates between different levels of danger:

- **DANGER**
Direct danger of severe injury or death
- **WARNING**
Possible danger of severe injury or death
- **CAUTION**
Risk of minor injuries
- **NOTICE**
Risk of extensive material/property damage

Miscellaneous symbols

These symbols are used in the document and on or in the unit:



Note, e.g. specific instructions regarding the efficient use of the unit.



Manufacturer



Distributor



Part number



Serial number



Health Industry Bar Code (HIBC)

1.2 Copyright information

All circuits, processes, names, software programs, and devices mentioned in this document are protected by copyright.

Any reprinting of the installation and operating instructions, in whole or in part, is only permitted with the written approval of the owner of the corresponding rights.

2 Safety

2.1 Transportation and storage

The original packaging provides optimum protection during transport and storage.

If required, original packaging for the unit can be ordered from Dürr Dental.




Dürr Dental will not accept any responsibility or liability for damage occurring during transport due to the use of incorrect packaging, even where the unit is still under guarantee.

- Transport the device in its original packaging only.
- Keep the packing materials out of the reach of children.

Product description

3 Overview

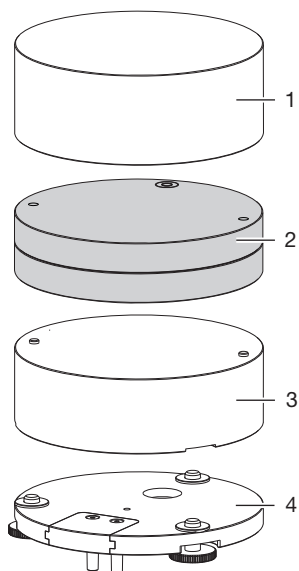

- 1 Upper absorber
- 2 Test phantom consistency check CBCT (not included in the scope of delivery, see "3.2 Accessories")
- 3 Lower absorber
- 4 Adjustment unit

3.1 Scope of delivery

The following items are included in the scope of delivery (possible variations may apply due to country-specific requirements and/or import regulations):

**Test phantom set for acceptance test
CBCT A7760**

- Upper absorber
- Lower absorber
- Adjustment unit

3.2 Accessories

The following items are required for operation of the device, depending on the application:
Consistency check test phantom 3D set . A7759



Usage

Refer to the x-ray's Installation Instructions for acceptance check procedure. Refer to the x-ray's Operating Instructions for consistency check procedure. Acceptance check must be done before consistency check.



Distributed by:

Air Techniques, Inc.
1295 Walt Whitman Road
Melville, New York 11747-3062, USA
Phone: +1 800-247-8324
Fax: +1 888-247-8481
www.airtechniques.com



Hersteller / Manufacturer:

DÜRR DENTAL SE
Höpfigheimer Str. 17
74321 Bietigheim-Bissingen
Germany
www.duerrdental.com

