

ProVecta 3D Prime / ProVecta 3D Prime Ceph

The latest version of the instructions is available in this page:



<https://bit.ly/m/air-techniques-imaging-manuals>

The following information supplements the installation and operating instructions of the unit.



As a general rule, the installation and operating instructions of the device must also be observed. These instructions include important information such as safety instructions and information on the setup, electrical connections, disinfection process, cleaning process etc.



WARNING

Risk of cross contamination due to non-reprocessed bite block

› Reprocess the bite block in accordance with the reprocessing instructions.



The bite block can be used with or without a hygienic protective cover.

If a hygienic protective cover is used, it must be replaced before each use.

If the bite block is used without a hygienic protective cover, it must be reprocessed in accordance with the validated reprocessing instructions



If the comfort bite block is used with a foam pad, the pad must be replaced before each use.

If the comfort bite block is used without a foam pad, it must be reprocessed in accordance with the validated reprocessing instructions.



Hersteller / Manufacturer:

DÜRR DENTAL SE
Höpfheimer Str. 17
74321 Bietigheim-Bissingen
Germany
www.duerrdental.com

Manufactured for / Distributed by:

Air Techniques, Inc.
1295 Walt Whitman Road
Melville, New York 11747-3062, USA
Phone: +1 800-247-8324
Fax: +1 888-247-8481
www.airtechniques.com