

Drain Tube and Drip Tray Installation Instructions

This document provides instructions to install the Drip Tray, P/N 31931, and 4 feet of 10mm tubing, P/N 54509, used to drain moisture collected in the Membrane Dryer during operation. These parts are provided with the AirStar compressor for the normal installation of the unit. Install the drip tray and drain tube by performing the following steps.

1. Place the Drip Tray under the push to connect connector located on the bottom of the Membrane Dryer
2. Install the 10mm tubing by inserting one end into the push to connect connector located on the bottom of the Membrane Dryer.
3. Place the unconnected end of the 10mm tubing into an open floor drain or the Drip Tray. Cut the tubing to the length as required when draining into the Drip Tray.

