

AIR TECHNIQUES

Clean Evacuation Lines Lead to a Safer Practice

Using integrated Vortex technology, the CleanStream Dispenser atomizes the Monarch CleanStream Evacuation System Cleaner through the inner tubing walls for a fast and effective cleaning routine

Dental professionals are not ones to leave safety to chance. But getting caught off guard by clogged evacuation lines in the office can put even the safest mindset into question. That's why keeping your lines clean and free of debris is so important in promoting patient safety, avoiding unexpected shutdowns, and ensuring ideal aerosol control. With the Monarch CleanStream Evacuation System Cleaner, you can do just that.

Natalie Glogowski, a dental assistant in Hinsdale, IL, has been using the Monarch CleanStream Evacuation System Cleaner for the past two years. After having a less than satisfactory experience with the office's previous product and seeing many positive reviews about CleanStream, she and her dental team decided to make the switch.

Dispensing a Safe Environment

The environmentally friendly and completely biodegradable Monarch CleanStream Evacuation System Cleaner is most effective when used with the CleanStream Dispenser. The Vortex technology simplifies and enhances cleaning by creating a mixture of air and liquid—atomizing the solution

through the lines and cleaning the entire inner tubing walls leading to the utility room.

Glogowski enjoys the dispenser's "mix, shake, and aspirate" approach to cleaning vacuum lines, as well as the fact that it can be easily carried between operatories without spilling. She shared that her office has not had any clogs in their lines since using it. "The CleanStream Dispenser is very easy to use with our lines," added Glogowski. "The CleanStream Dispenser speeds up our line cleaning process because we can leave it and do other things while we let the dispenser water get sucked through the lines."

Clean evacuation lines that are free of debris encourage the flow needed to effectively mitigate aerosols with high-volume evacuation, especially when handpieces or ultrasonic scalers are in use. CleanStream is designed for daily cleaning, which benefits practices by ensuring regular cleansing and removal of line buildup. Having cleaner lines gives Glogowski more confidence that aerosols are being properly reduced.

"Because our lines are cleaner and deodorized, thanks to the CleanStream, there is less worry that the germs being produced by patients will get into the air and contaminate the space," said Glogowski.



Natalie
Glogowski
Hinsdale, IL

"The CleanStream Dispenser speeds up our line cleaning process because we can leave it and do other things while we let the dispenser water get sucked through the lines."

PRODUCT FEATURES

- Unique formula designed to help remove line buildup and maintain vacuum performance and suction
- Cleans evacuation lines for all wet and dry vacuum systems
- Lasts up to 5 days if pre-mixed
- Formula is biodegradable and compliant with BMP for dental amalgam waste